




Types of fine bubble



Features of the fine bubble

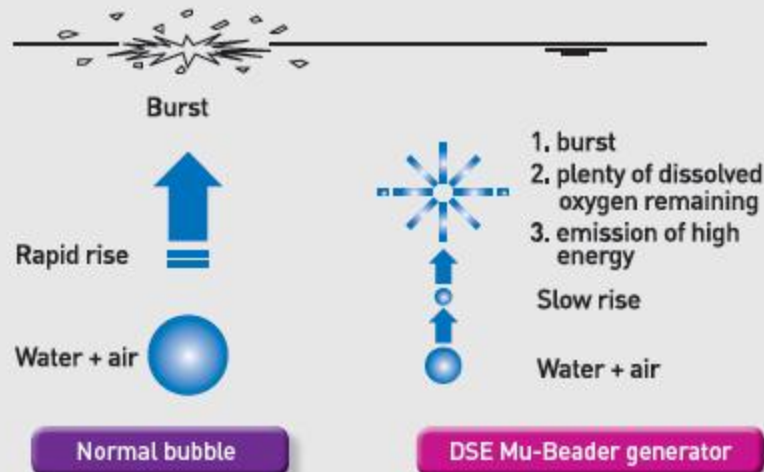

Intended purpose and specifications

Use	for medical devices with an electric motor that is designed for the absorption of sebum and acnes
Electric rating	1. Voltage : 220VAC 50/60Hz 2. Power consumption : 300VA
Physical	1. Dimension : 400(D) X 400(L) X 1,000(H) mm 2. Weight : 35Kg
Medical suction system	1. Handpiece tip : plastic, silicone 2. Vacuum pressure : within 250~600mmHg±10%



DSE Overseas marketing : D.S.E.

Address: #105, Byok san 3 cha Digital valley 212-13, Guro-3Dong, Guro-Gu, Seoul Korea
Tel : 82-2-2025-1151 Fax : 82-2-2025-1154 E-mail : dsemed@dsemed.com



Refreshing energy*

will make your skin and scalp much cleaner and whiter.

Beading Action Therapy System
Mu: Beader™



Removal of horny substances and supply of moisture at the same time*

Mu: Beader™ is designed to promote exfoliation with fine bubbles to remove aged dead skin cells and sebum and, at the same time, inject highly concentrated, nutritious serum so as to make skin and scalp moist, smooth, and lustrous.

Safety: neither downtime nor side-effects*

Because treatment does not cause any damage to skin and scalp, patients can lead a normal life right after treatment. There are no side effects such as injuries, pain, burns, rashes, etc.

Multiple skincare effects*

Treatment removes foreign materials (such as cosmetics and sebum) so users can expect to achieve whitening and pore-tightening effects.

All the more clean and bright with refreshing energy in our skin and scalp!!!

Mu: Beader™ is...

not a previously available decortication, but a new-concept natural spa skincare system, which enables you to painlessly achieve in a comfortable environment a variety of treatment ranging from cleansing of skin to removal of obsolescent horny substances to extrusion of sebum to supply of nutrition simultaneously. You will feel a moist and refreshing sensation after treatment.

User Manual for Mu: Beader™

Mu: Beader™ adopts a vacuum extrusion method. The hand piece should come into close contact with the skin. You can control pressure for both fine bubbles and serum liquid to perform treatment. Red areas on the skin that may be created during operation will disappear normally in 20-30 minutes and the skin restores healthy and lustrous condition.

Mu: Beader™ Name and function of parts and components



- AG Supply of serum fluid and removal of horny substances and body wastes
- BG Supply of serum liquid and removal of horny substances and body wastes
- AO Serum supplies nutritional elements to skin

Control of the intensity of suction according to skin sensitivity

1. Handpiece for AG, BG, AO
2. Handpiece for fine bubble therapy

01 Cleansing

wipe wastes 2-3 times off the surface of the face with a cleansing cream

02 Diagnosis of scalp and skin and Selection of serum

identify whether the skin is normal, oily, sensitive, or of acne type and make selection of a serum suitable for the skin type.
[AG serum: for normal skin; BG serum: for acne or sensitive skin]

03 Treatment (removal of keratin and sebum)

first spread the selected fine bubble and serum over the face and then divide the face into 3-5 areas to start treatment for about 2 minutes.

04 Washing

wash and wipe out the face cleanly with a spongy after treatment.

05 Treatment (supply of moisture and nutrients) and finish

perform treatment while letting AO serum penetrate into the skin; after treatment, give a finishing touch with a skin balm or a nourishing cream.



In the order of Neck >>> Chin >>> Cheek >>> Forehead >>> Nose

1. Neck: after turning face a little sideway, make lymph nodes relax by massaging the area from top to bottom and in the opposite way with the sternocleidomastoid region in the center.
2. Nose: perform treatments repeatedly because there is lots of sebum in this region.
3. Perform treatment by traveling all across the face from bottom to top along the texture of muscles.
4. Treatment is recommended to be made for about 2 minutes respectively in the order of the neck → chin → cheek → forehead → nose.

Special tips

For removal of keratin as well as for extrusion



The special tip of the Mu: Beader™ consists of two suction holes and one serum discharge hole. Its specially-designed spiral tip is intended to hold a product long enough to soften dead cells of the scalp and skin, provide nutrients, and absorb them in a vacuum state to maximize its peeling and extrusion effects.

Mu: Beader™ Serum



AS, S1 [For all Skin Types- For the general skin, 500ml]

Since glucosamine and lactic acids promote the peeling of the skin to remove impure substances softly and plant extracts soothe and moisten the skin smoothly, the product enables a scientific treatment that does not produce any injuries or rashes while achieve a bright and healthy-looking skin regeneration.

BG, S2 [For oily and acne- For oily and acne skin, 500ml]

Salicylic acids and lactic acids get rid of acne-causing bacteria (Propionibacterium acne) and other body wastes and natural moisturizing substances provide even a sensitive skin with excellent soothing effects.

AO [Nutrition and supply of moisture for all types of skin, 500ml]

The deep penetration of vitamin C, E, Aloe Vera, and green tea extract will help skin to restore and regenerate damaged skin cells to maintain a moist and bright skin tone without any side effect.