



3 IN 1 SYSTEM

HIGH POWER

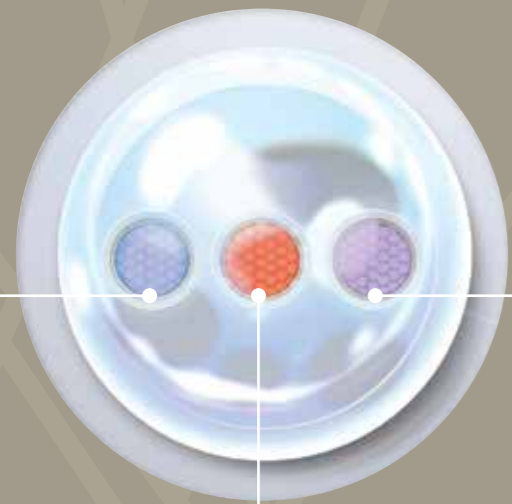

HEAD	UV(415nm)	RED(635nm)	IR(830nm)	Excerpt
INTENSITY (MAX)	100mW/cm ²	115mW/cm ²	330mW/cm ²	Korea medical device evaluation institute

< INTENSITY(MAX) >

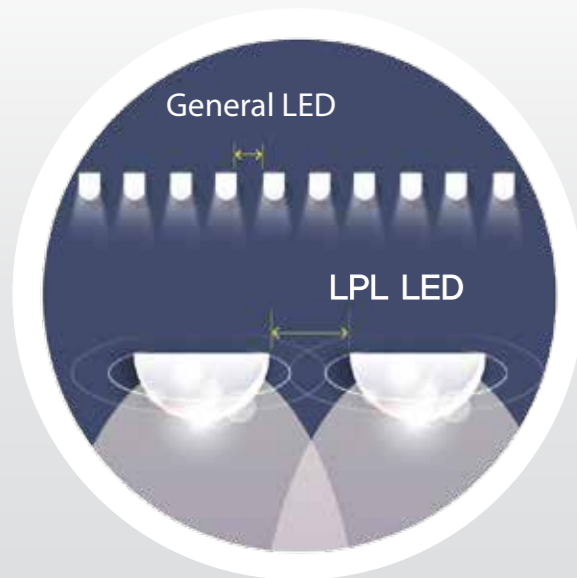

HIGH POWER LED

World's First!

Treatment methods


High power 3 in 1 system

LPL PDT GEL photo Q



1. Cleansing (Aqua Peeling / deep cleansing, etc.)
2. Acne extraction
3. Coating
4. Light blocking for 10-15 minutes
5. 635/415 10-15 minutes

LPL soothing regeneration Hydro Regen

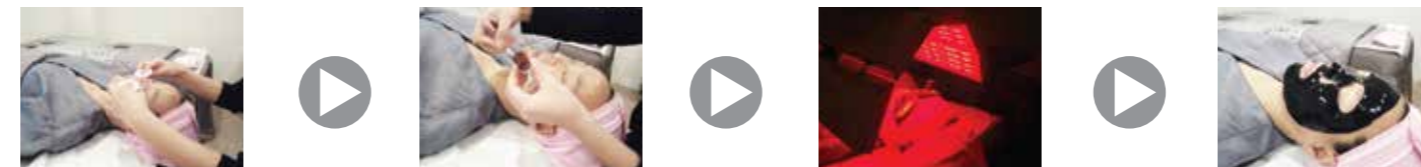

1. Cleansing (Aqua Peeling / deep cleansing, etc.)
2. LPL Hydro Regen coating
3. 10- 15 minutes LPL
4. Mask pack / soothing pack

LPL Features

High power 3 in 1 system

- Three wavelengths (415/635/830) implemented in a machine
- Using high-efficiency lens
- High output LED Power
- Long durability
- User's power adjustment according to the patient's condition
- 9 functions of memory
- Low noise
- Dedicated solutions


LPL Solutions

High power 3 in 1 system


PDT Gel Photo-Q

The high-energy of chlorophyll activated by LED kills acne and releases from one's body.


Acne bacteria killing mechanism by photodynamic therapy (PDT) of Photo Q

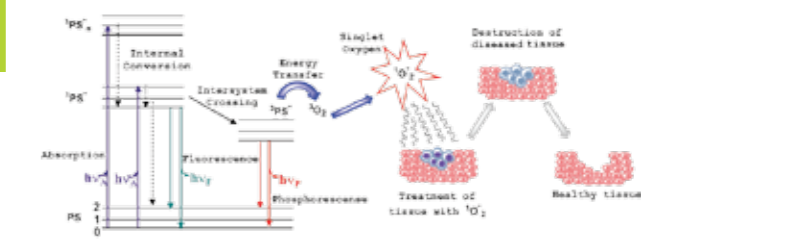


Step 1: Activation of the PS (chlorophyll) by LED

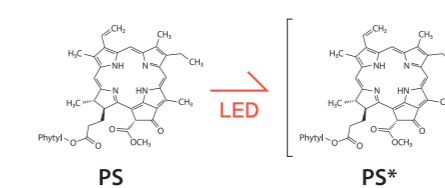


Step 2: Activated chlorophyll transforms to the energy of oxygen

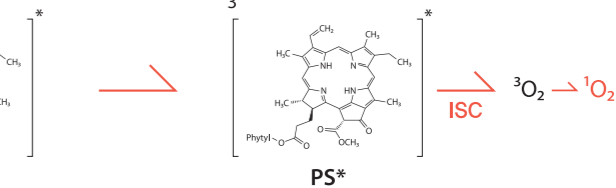

Step 3: Acne is attacked and killed by the high energy 1O2

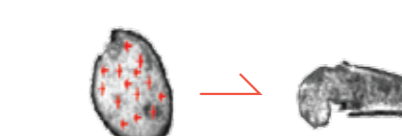


Step 4: Release of chlorophyll from one's body

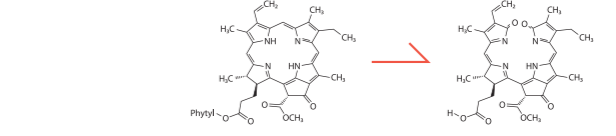





Hydro Regen for Rejuvenation

The stem cell ingredients, that are extracted from orchid, tomato and rice and cultured, improve the skin's resilience after the medical treatment, helps rapid regeneration by supplying growth factors to red and damaged skin, and soothes the skin comfortably with adequate moisture supplies.

Scanning Electron Microscope (SEM) Image of Phyto StemCell Complex

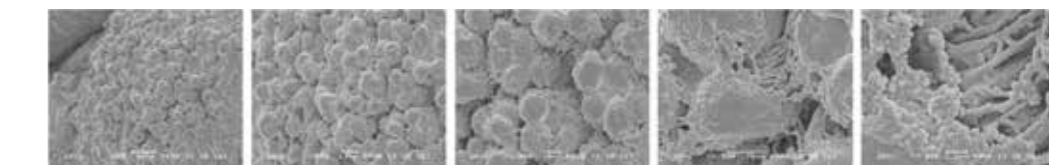

Anti-wrinkle effect

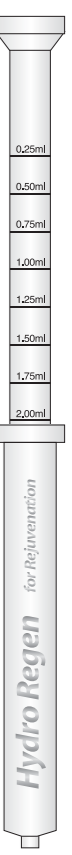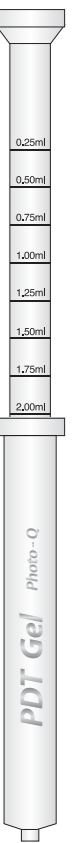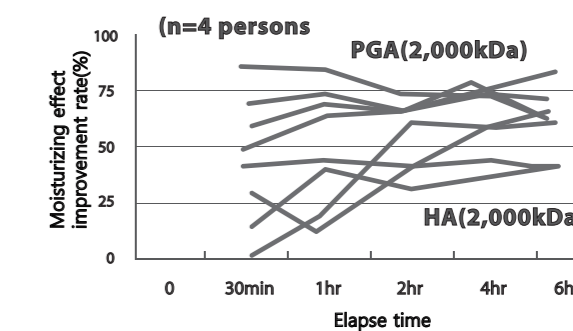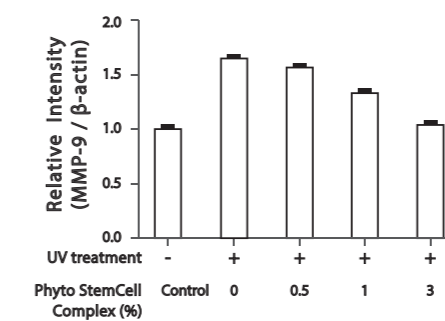

Light energy
therapy effect
that has not been
experienced so far.

LPL
LED Pulsed Light



High power 3 in 1 system



415.635.830nm LED LED Medical Device / Medical device that may be used for 10 years

LPL Sales guide

Only for internal use (in-house training)

ids

Innovation for better life

Address #105 Byoksan digital Valley 3 cha, 271, Digital-ro, Guro-gu, Seoul, Korea

Tel +82 2 2025 1150-3 Fax +82 2 2025 1154

New E-mail info@ids.co.kr / www.ids.co.kr Old E-mail dsimed@korea.com / www.dsemed.co.kr

For more information about international buyers, please contact us via email

